

Patient Positioning Techniques

Exam Table Methods

1



Preferred position - side*

2



Alternative position - side*


3



Alternative position - prone

Chair Method

4

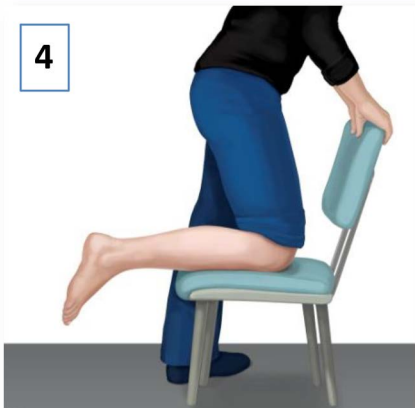

Alternative position - leg up



Check Your Position

Tester:

- Patient's outer ankle bone and Achilles tendon are visible
- Stable grip on device

Patient:

- Fully relaxed upper and lower leg
- Conditions above not met? Adjust position or try a different one.

The illustrations above demonstrate the recommended patient positions for testing with the DPNCheck device. Patients must be in a comfortable position that allows for relaxation of the leg and foot. Please call Customer Support at +1 844.DPN.CHECK (1.844.376.2432) with any questions.

*Test leg on top.