


Sural Nerve Conduction Interpretation Guide

*This guide is for informational purposes only.
It does not provide a clinical diagnosis.*

Nerve Conduction Reference Ranges*

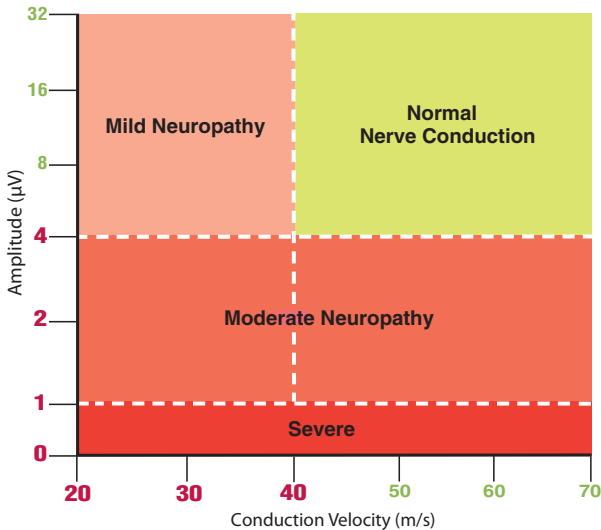

Severity	Amplitude	CV
Normal	Normal	Normal
Mild	Normal	Abnormal
Moderate	Abnormal	Normal or Abnormal
Severe	Undetectable	—

*Data on file

Error Code Key†

Display	Light	Description
Lb	●	Incorrect limb selected
Pn	●	Data quality issue requiring retest
Pr	●	Stimulation issue. Possible causes: <ul style="list-style-type: none"> • Insufficient gel on probes • Poor probe contact with skin • Inadequate skin preparation • Probe movement during test • Gel smearing between probes
Sn	○	Incorrect biosensor insertion or defective biosensor
Sn	●	Data quality issue. Possible causes: <ul style="list-style-type: none"> • Biosensor backing not removed • Incomplete biosensor contact • Skin inadequately prepared
Ec	●	Data quality issue. Possible causes: <ul style="list-style-type: none"> • Excessive muscle contraction or patient movement • Biosensor backing not removed
oC	●	Patient ankle too cold
Lo	●	Low battery
Hd	●	Device hardware issue
Ub	●	Excessive biosensor reuse detected, device is disabled

†Please refer to the Reference Guide for a detailed list of causes and solutions.